2797

ASTER

KIYOMI
estética dental**DENTSCARE LTDA.**

Av. Edgar Nelson Meyer - 474 - Distrito Industrial
CEP 89.219-501 - Joinville - SC - Brasil
Fone: +55 (47) 3441.6100

CNPJ/TAX ID: 05.106.945/0001-06
INDÚSTRIA BRASILEIRA/BRAZILIAN INDUSTRIY
Resp. técnico/Technician in charge:
Friedrich Georg Mittelstädt CRQ 13100147-SC
Made in Brazil - Marca/Brand: FGPM - 55532

EC REP NUNO FLORES

Al. Bonifácio Lazaro Lozano, 3 - Piso O - C
2780-125 Oeiras / Portugal - Tel: (351) 21 4439292
ec.rep@fgm.ind.br

Servicio al profesional
984 491 808

hola@lpgdental.com

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Read all information in this instruction manual before using the product. Save it for consultation, at least until the product has been totally used and/or there is no more interaction of the product with its last patient.

Product description

ASTER is a radiopaque, light-curing composite resin with fluorescent and opalescent properties, high polishing power and several options of shades and translucence. It is an aesthetic restorative material used for restorations of anterior and posterior teeth. ASTER was carefully developed to reproduce the polychromatism of teeth in great detail, with great mechanical properties.

Presentation

1 syringe with 1 resin shade.
Shades: EA1, EA2, EA3 and DA1, DA2, DA3 and B0.5.

Basic composition

ASTER is a composite resin composed by a monomeric matrix containing Bis (GMA), Bis (EMA), UDMA and TEGDMA. The fillers are a careful combination of Barium-Aluminum, Silanized Silicate and particles of Silicon Dioxide, Camphorquinone as Photoinitiator, accelerators, stabilizers and pigments. The composite has particles in the range 40 nm to 3.0 microns with an average particle size of 0.5 microns, the total filler content of 78.5% to 79.8% in weight and 57% to 58% in volume of inorganic fillers. The distribution of the sizes of the particles allows an adequate filling of the resin which contributes for its high mechanical strength as well as resistance to wear. Such characteristics are necessary for restorations of posterior teeth and for easy polishing, generating a restoration with flat surface and high brightness for anterior teeth.

Product indication

The composite is suitable for use in permanent and deciduous teeth:

- Direct restorations in anterior and posterior teeth
- Classes I, II, III, IV, V and VI.
- Direct veneers with composites.
- Cementation of tooth fragments.

- Core build-ups.
- Teeth splinting.
- Diastema closing or reduction.
- Modification of teeth's shape (e.g.: conoid teeth).
- Porcelain/composite repairs.

ASTER shade system

ASTER uses the VITA® Classical as guide for its shades of enamel and dentin.

- The ASTER resin must be light cured layer by layer with a blue light and with a minimum intensity of 400 and 500 nm with at least 450 mW/cm². Do not remove the inhibiting layer as this affects the adhesion of the following layer of resin.
- The shade B0.5 can be used in pediatric dentistry reproducing details of the deciduous teeth anatomy.

Table 1: Availability of shades, quantity, light curing time and translucence

Type	Shades	Quantity	Light curing time*	Translucence (%)
Dentin	DA1	4g	40s	43-46
	DA2	4g	40s	43-46
	DA3	4g	40s	43-46
	B0.5	2g	60s	34-36
Enamel	EA1	4g	20s	53-56
	EA2	4g	20s	53-56
	EA3	4g	20s	53-56

The table 1 presents all shades of ASTER resin available, their respective translucence, commercialized amounts and light curing time.

- The light curing time described on the table 1 must be followed to guarantee the curing degree, conversion degree, shade stability and mechanical properties of the resin. The capacity of the light to go through the resin is associated to translucence. As the ASTER resin presents different degrees of translucence, several studies were accomplished to determine the appropriate light curing time in order to achieve the resin properties and guarantee its quality. In case of insufficient exposure, the resin properties may not achieve the ideal values. Overexposure is not a possibility.

Precautions and contraindications

This product is intended to be used only according to this instructions manual.

Do not use the product if the patient is allergic to one of the substances present in the composition. In case of allergic reactions, discontinue the use of the product and, if needed, seek medical attention. Avoid the contact of uncured resin with the skin, mucosa and eyes because it may present an irritating effect due to the presence of methacrylates. It is recommended to wear all the Personal Protection Equipment (PPE) while manipulating the product. Avoid the use of eugenol containing materials, such as liners or temporary restoratives, as they hinder proper light curing of the product. When restoring deep cavities, use material (such as calcium hydroxide) in contact with the pulp wall. The composite is sensitive to light, so make sure to keep the syringe properly closed when not in use. The resin content should be used right after being extruded from the syringe. Use protective goggles when light curing the composite. The product must not be swallowed or aspirated. If so, seek medical attention. The product is contraindicated when a dry.

Side effects
This product contains Methacrylate Monomers which may cause slight allergic or irritant reactions on patients who are sensitive to such substances.

Instructions for use
Read carefully the Warnings, Precautions, Contra-Indications and Possible Side Effects written in this Product information sheet, before the beginning of the treatment.

Shade taking

- Before taking the shade of the tooth to be restored, it must be clean and hydrated, without isolation. Prophylaxis is highly recommended in this step.
- The VITA® Classical shade guides may be used for shade taking.
- Alternatively, a small portion of the composite may be placed over the surface of the tooth to be restored (without etching nor application of an adhesive system), light cured for 30 seconds, humidified with the patient's saliva and observed. This stage may be accomplished to confirm the shade of the composite.

Product application

1. Isolate the operative field. The isolation must guarantee a humidity- free area which is clean and visible during the whole treatment. If necessary, perform the absolute isolation.
2. Prepare for the execution of direct adhesive restorations.
3. Acid etch (Phosphoric Acid of 37%) the margins of the enamel initially and then inside the cavity. The acid etching time is 15 seconds. After that, wash with water abundantly, remove all the acid from the tooth structure and dry the dentin without dehydrating it.
4. Apply the adhesive system according to the manufacturer's instructions. If using a self-etching adhesive (por ejemplo K-BOND+ Universal, KIYOMI) skip step 3.
5. Insertion and light curing of the composite resin:
 - We recommend the technique of insertion and light curing by increments. Such technique allows a more uniform light curing. Follow the time described in the table for layers of up to 1,5 mm of resin.
 - Restore the cavity according to the selected shade(s) applying small layers of ASTER (maximum thickness of 1,5 mm) adapting them in the cavity carefully.
6. For finishing and polishing, we recommend sand disks and polish pastes with the aid of felt disks.

Storage and maintenance
The product should remain in its original packaging and be kept tightly closed.
Protect from direct sunlight. Store at temperature between 5 °C y 30 °C / 41 °F and 86 °F. Do not freeze the product.

Expiration date
Printed on the product packaging.

Warnings
Dental products should be used by authorized professionals only. Do not use the product after its expiration date. Follow your country's legislation when disposing the product. Keep out of the reach of children.

This material was manufactured for dental use only and must be manipulated according to the manufacturer's instructions of use. The manufacturer is not responsible for any damage caused by misuse or improper manipulation of the product. The user is obligated to prove (before using the product and under his/her responsibility) that this

product is compatible with the desired utilization, especially when such utilization is not indicated in the instructions of use. Data description does not constitute any type of warranty and therefore is not binding.

Leyenda de símbolos / Meaning of symbols	
	Fecha de fabricación Date of manufacture
	Número de lote Batch Number
	Fecha de validez Shelf life
	Limites de temperatura Temperature limits
	No lo utilice si el envase está dañado Do not use if packaging is damaged
	Consérvelo seco Keep it dry
	Manténgalo lejos de la luz solar Keep protected from sunlight
	Fabricante Manufacturer
	Consulte las instrucciones de uso Refer to instructions for use
	Representante en la Comunidad Europea Representative in the European Community
	Marcas CE para comercialización en la Comunidad Europea CE Mark for European Community market