DENTSCARE LTDA.

Av. Edgar Nelson Meister - 474 - Distrito Industrial
CEP 89-201-501 - Joinville - SC - Brasil
Fone: +55 (47) 34416100
CNPJ/TAX ID: 05.106.945/0001-06
INDÚSTRIA BRASILEIRA / BRAZILIAN INDUSTRY
Resp. técnico/Technician in charge:
Friedrich Georg Mittelstädt: CRM 1310047-03
Made in Brazil - Marca/Brand: FGM - 55536

EC NUNO FLORES

Al. Bonifácio Lázaro Lozano, 3 - Piso 0 - C
2780-125 Oeiras / Portugal - Tel: (351) 21 4439292
ec.rep@fgm.ind.br

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984 491 808

hola@ipgdental.com

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EN Instructions for use

Flowable light-curing composite resin Professional use only

Read all information in this instruction manual before using the product. Save it for consultation, at least until the product has been totally used and/or there is no more interaction of the product with its last patient.

Product description

ASTER Flow is a light-curing microhybrid composite resin of medium viscosity used for the restoration of slightly invasive preparations, pit and fissure sealant, base/lining underneath direct restorations, in tunnel-type preparations, radiopaque lining of cavities, repairs of enamel defects, restorations of primary teeth, repairs in composite resin, bonding of tooth fragments, class I, III and V restorations, non-carious cervical lesions.

ASTER Flow can be used separately or with ASTER and most of the light-curing composites.

ASTER Flow is light-cured with blue wavelength light between 400 and 500 nm (blue light) and with at least 450 mW/cm².

Presentation

1 syringe containing 2g and 5 applicator tips, available in the shades: A1, A2, A3 and B0,5.

Basic composition

Methacrylic Monomers such as Bis (GMA) Monomer, Bis (EMA) e TEGDMA, Canforquinone, co-initiators, conservants, pigments and 72 % of Silanized inorganic load composed of microparticles of Barium-Aluminum-Silicate and nanoparticled Silicone Dioxide with the particles in sizes between 0,05 and 5,0 microns.

Product indication

- Base/lining underneath direct restorations.
- Small, non occlusal stress-bearing class I restorations according to minimally invasive filling therapy.
- Pit and fissure sealant.
- Tunnel-type preparation.
- Repair of enamel defects.
- Bonding of tooth fragments.
- Repairs in composite resin.
- Non-carious cervical lesions.
- Planning of preparation walls.

Precautions and contraindications

- For dentistry use only.
- Read carefully the recommendations and instructions for use.
- Use ASTER Flow only as informed in these instructions.
- Do not use ASTER Flow in patients with a history of allergy to composite resins that are Acrylate based. In case of allergy, suspend the use and request the patient to seek medical assistance.
- In order to minimize the risk of irritation, regions of the cavity preparation too close to the pulp should be adequately protected (e.g.: application of Calcium Hydroxide).
- To avoid allergic reactions to Methacrylate Monomer, it is recommended to constantly wear gloves. If ASTER Flow gets in contact with the hands, wash them thoroughly with water.
- In case of eye contact, wash them thoroughly with water and contact a doctor immediately.
- When using a light-curing device, eye protection should be worn.
- ASTER Flow is contraindicated when it is not possible to establish dry operative field or it is not possible to follow the described technique correctly.
- Eugenol and Hydrogen Peroxide containing materials should not be used in conjunction with this product because they may interfere with hardening and cause softening of the polymeric components of the material.
- ASTER Flow should not be swallowed or inhaled. In case this occurs, seek medical help immediately.
- The enclosed intraoral tips are intended for a single use and must be disposed of after only one use. Return the cap to the syringe after use.
- Contact with saliva and blood during composite placement may cause failure of the restoration.

Use of rubber dam or adequate isolation is recommended.

Side effects

- The product may cause some level of irritation if in contact with the dental pulp or too close to it. In cavities that are too deep, use a lining material with calcium hydroxide base.
- In cases of allergic reactions to the product, suspend its use. ASTER Flow is contraindicated for patients that are allergic or hypersensitive to Methacrylate and related Monomers and should not be used in those patients.
- Product may irritate the eyes and skin.
- Eye contact: irritation and possible corneal damage. Skin contact: irritation or possible allergic response. Reddish rashes may be seen on the skin.

Instructions

Application in direct restorations:

1. Shade taking: take the shade with the teeth well hydrated.
 2. Preparation of the cavity: the preparation of the cavity should be accomplished according to the norms of the adhesive technique, removing only the carious tissue, there is no need for additional retentions.
 3. Pulp protection: cover only the areas very close to the pulp (e.g. Calcium Hydroxide).
 4. Etching of enamel and/or dentin: apply Phosphoric Acid of 37% on the enamel and/or dentin and let it act for 15 seconds. Soon after, wash abundantly with water and dry with oil free air jets. Avoid the dehydration of the dentin.
 5. Adhesive agent: apply an adhesive system according to the manufacture's instructions. If using a self-etching adhesive agent (e.g. K-BOND+ Universal, KIYOMI) it is dispensable the step 4.
 6. Application of ASTER Flow: apply the material directly in the cavity with the application tip. Each applied layer should not be thicker than 2 mm. Light-cure each layer for 40 seconds.
 7. Finish and polishing: use disks, felt disks and polishing pastes.
- For pit and fissure sealant:**
1. Make the total isolation and the acid attack of the pit and fissure surface, wash and dry.
 2. Apply ASTER Flow with the aid of an application tip. Do not allow the product to spread to regions of occlusal contact.
 3. Light-cure for 40 seconds. Accomplish the finishing and the polishing recommend for light-cure resins.

Conservation and storage

- Keep out of direct sunlight, exposure to light leads to premature curing.
- Allow material to reach room temperature prior to use.
- Store in a well ventilated place at temperatures between 5 °C and 30 °C / 41 °F and 86 °F.
- Do not freeze.
- Protect from moisture.
- Do not use after expiration date.

Expiration date

Printed on the product packaging.

Warnings

- Any dental product should only be used by dentists, prosthetic technicians and authorized dental laboratories.
- Do not use the product if it is expired.
- For disposal, follow the legislation of your country.
- Keep it out of the reach of children.
- Avoid contact of non-cured ASTER Flow with skin, mucosae and eyes.
- Before light-curing, the product may have a slight irritating effect and promote hypersensitivity due to Methacrylates.

This material was manufactured for dental use only and must be manipulated according to the manufacturer's instructions of use. The manufacturer is not responsible for any damage caused by misuse or improper manipulation of the product. The user is obligated to prove (before using the product and under his/her responsibility) that this product is compatible with the desired utilization, especially when such utilization is not indicated in the instructions of use. Data description does not constitute any type of warranty and therefore is not binding.

Leyenda de símbolos / Meaning of symbols	
	Fecha de fabricación <i>Date of manufacture</i>
	Número de lote <i>Batch Number</i>
	Fecha de validez <i>Shelf life</i>
	Límites de temperatura <i>Temperature limits</i>
	No lo utilice si el envase está dañado <i>Do not use if packaging is damaged</i>
	Consérvelo seco <i>Keep it dry</i>
	Manténgalo lejos de la luz solar <i>Keep protected from sunlight</i>
	Fabricante <i>Manufacturer</i>
	Consulte las instrucciones de uso <i>Refer to instructions for use</i>
	Representante en la Comunidad Europea <i>Representative in the European Communit</i>
	Marca CE para comercialización en la Comunidad Europea <i>CE Mark for European Community market</i>