K DESENS 5%

1 jeringa desensibilizante con
nitrato de potasio 5%

Dentalsur2003 S.L.

Avda. Ortega y Gasset 152

29006 Málaga (Spain), +34 952 040 502

hola@ipgdental.com www.ipgdental.com

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984 491 808

hola@kiyomidental.com

Rx Only

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Desensibilizante

Solo uso profesional

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K **DESENS** **5%**

Desensitizing syringe with
5% potassium nitrate

DentalSur2003 S.L.

Avda. Ortega y Gasset 152

29006 Málaga (Spain), +34 952 040 502

hola@ipgdental.com www.ipgdental.com

Customer Service

+34 984 491 808

hola@kiyomidental.com

Rx Only

EN Instructions manual

Desensitizer

For professional use only

Please read all the information in this instruction manual carefully before starting to use.

Description

This kit contains a 3 ml syringe of desensitizing gel with 5% potassium nitrate, designed to alleviate dental sensitivity. This product is presented as an effective solution to reduce discomfort associated with sensitive teeth.

Presentation

3 ml syringe and tip for application.

Composition

Aqua, potassium nitrate, xylitol triethanolamine, carbopol

Product indication

Indicated for the treatment of teeth with hypersensitivity, both pre-existing to dental whitening and post-treatment.

Precautions

For external dental use only.

Keep out of reach of children.

Avoid contact with eyes; rinse with water if it occurs.

Do not use in case of allergy to any of the ingredients.

Side Effects

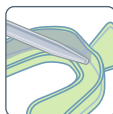
- Ingesting large amounts of the product can cause nausea and poisoning. In rare cases, gingival or oral irritation may occur.
- Occasionally sensitivity may require treatment. If the sensitivity becomes uncomfortable, treat the teeth with a soothing desensitising gel. The more concentration of peroxide, greater the possibility of suffering dental sensitivity.
- Areas of hypocalcification can exist which may not be visible to the naked eye. Hypocalcified areas will whiten faster, thereby becoming more obvious during bleaching. In some cases these stains disappear owing to the dehydration of the enamel after concluding the bleaching. Treating these areas with fluoride can also help to make them disappear.
- Some users can feel slight irritation in the gingiva, tongue or lips, generally caused by excessive use of gel in the tray.

If any adverse symptoms are experienced, discontinue use and consult a healthcare professional.

Instructions for use

1. Brush your teeth: Clean your teeth with your regular toothpaste.
2. Apply desensitizing gel to the dental tray: Remove the syringe cap and apply a small amount of gel to the provided dental tray.
3. Keep the tray in your mouth for 30-40 minutes: Gently place the dental tray in your mouth, ensuring that the gel is in contact with the sensitive teeth.
4. Remove the tray and rinse: After the recommended time, remove the tray and rinse your mouth with warm water to eliminate any gel residue.

Repeat this procedure daily until the syringe is empty. If sensitivity persists, consult your dentist.

Preservation and Storage

Store the product at 5°C and 25°C / 41°F and 77°F. Do not freeze the product. Keep it out of direct sunlight.

Warning

Do not use the product after expiry date. Maintain it out of reach of children. The discard of product follows the directive 94/62/CE.

This material was only manufactured for dental use and must be handled according to the instructions for use. The manufacturer is not responsible for damage caused by other uses or incorrect handling. Besides this, the user is obliged to prove, before use and under his or her responsibility that this material is compatible with the use desired, mainly when this use is not indicated in the instructions for use. Data descriptions do not constitute any type of guarantee and so are not in way linked.

Legenda de símbolos / Meaning of symbols

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